



## CEDARS-SINAI MEDICAL CENTER®

### Instructions for Blood Collection - INFANT/SMALL CHILD

1. Collect the following tubes on the above named person(s). Please label each tube with the person's name, date of birth and the date drawn.

- 1 Purple top tube (EDTA)
  - 1 Yellow top tube (ACD)
- Minimum of 3-5 cc's per tube

2. Samples should be placed in absorbent packaging and closed securely.
3. Please be sure tubes are secured and in packaging so they will not break during transport.
4. All blood can be shipped at room temperature.
5. Place packaged tubes into express mail package, and send **NEXT DAY** delivery to the following address:

Arleen Hernandez  
Medical Genetics Institute  
Cedars-Sinai Medical Center  
Pacific Theatres Bldg.  
International Skeletal Dysplasia Registry  
116 N. Robertson Blvd. 4<sup>th</sup> Floor  
Los Angeles, CA 90048  
Phone: (310) 423-9915

Please express mail samples the same day as collected for delivery the next day.  
***Special Circumstances:***

**If a sample is drawn on a Friday or the day before a holiday, please contact us to make special arrangements for delivery.**

**The purple top tube should be refrigerated until shipped.**

**The yellow top tubes should be stored at room temperature until shipped.**

**Express mail samples on Monday or the next business day.**

6. If you or the healthcare professional collecting your blood has any questions, please call our office at (310) 423-9915.